




Easyboot ZIP

The Easyboot Zip is a simple slip-on boot designed to protect the hoof. Safe and easy to apply, the Zip is a better alternative to clumsy foot wraps for medicating the hoof or keeping the hoof free from dirt and oils. The removable internal leather sole helps pull moisture from the hoof and can be easily removed for cleaning.

- Protects the hoof from dirt and oils
- Safe and easy to apply
- A better alternative to awkward foot wraps
- Allows the hoof to breathe
- Keeps the hoof clean
- Ideal for medicating the hoof
- Removable leather sole insert
- Machine washable
- Available in 5 sizes

*Provides a clean, protected environment for the hoof.
Ideal protection for a variety of uses. Comes in 5 sizes.*

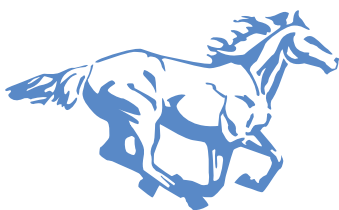

easycareinc.com

EasyCare

blog.easycareinc.com

How to Contact Us

2300 E. Vistoso Commerce Loop Rd. Tucson, AZ 85755
Phone: 800.447.8836 • Fax: 520.297.9600

easycareinc.com

admin@easycareinc.com


facebook.com/easyboot

youtube.com/easycarevideos



How to Measure for Boots

1. After a fresh trim, measure the width of the hoof across the sole at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.

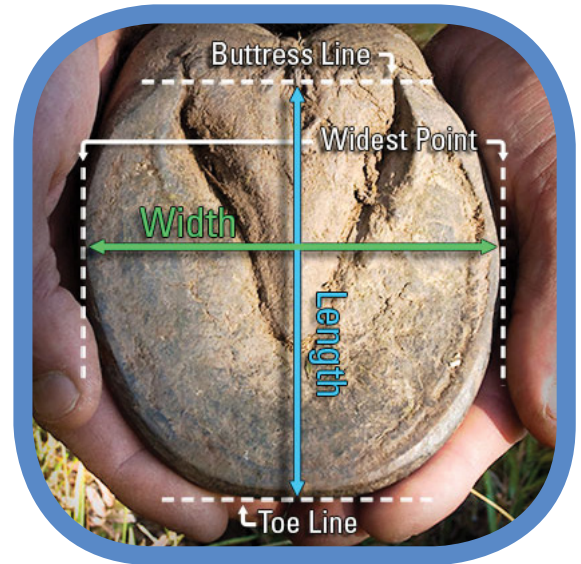

3. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, this boot style is not recommended. Be sure to check your measurements against an alternative boot in the EasyCare line--another style may accommodate your horse's hoof shape better.

Size	Width in inches	Length in inches
0	3 - 3 13/16"	3 - 3 13/16"
1	3 7/8" - 4 11/16"	3 7/8" - 4 11/16"
2	4 3/4" - 5 3/8"	4 3/4" - 5 3/8"
3	5 7/16" - 6 3/8"	5 7/16" - 6 3/8"
4	6 7/16" - 7 1/16"	6 7/16" - 7 1/16"

Size	Width in mm	Length in mm
0	77-97mm	77-97mm
1	98-119mm	98-119mm
2	120-136 mm	120-136 mm
3	137-162 mm	137-162 mm
4	163-180 mm	163-180 mm



easycareinc.com
EasyCare
 blog.easycareinc.com

How to Contact Us

2300 E. Vistoso Commerce Loop Rd. Tucson, AZ 85755
 Phone: 800.447.8836 • Fax: 520.297.9600